

Ear Impression Process

1. Enquire.
2. Check ears with speculum if available.
3. Place cotton approximately 2-3mm past the second bend.
4. Prepare the silicone mold material. Be sure to watch the time lest the material hardens.
5. With your left hand, secure the string attached to the cotton swab while injecting the mold into the ear, starting from deep within the canal.
6. Allow impressions to dry for 5 minutes.
7. Once dry, gently pull the ear upward to release air pressure and slowly rotate the dried ear impression forward until it's released from the ear. Check to make sure the impression quality looks good.
8. Check the impressions for residue and wax. Clean the impressions as needed.
9. Place the impressions in a secure bag and be sure to mark it with your order number.

Getting Ear Impressions Done is Not Recommended When:

- A. You notice any inflammation or discharge from you ears
- B. If you've recently had otitis media (middle ear infection), perforated eardrum, or surgery.
- C. You have an occluded ear canal due to ear wax buildup.
- D. There is not a reliable center to obtain your ear impressions.